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Art.-Nr. 78072 000

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Bathtub safety rail Deluxe Plus

Item No. 78072 000

Please read these notes carefully before use. Be sure to follow the instructions and keep them in a safe place!

The Bathtub Safety Rail Deluxe Plus makes it easier to bathe without assistance. It serves as a handgrip when you are climbing over the side of the bath – see fig.1. Using the Safety Rail helps you to keep your balance, making bathing feel much safer, especially where wet and slippery surfaces can make you feel uncertain.

⚠ Safety instructions ⚠

- Before you use the Bathtub Safety Rail, make sure that it is securely fixed in place – every time!
- The Bathtub Safety Rail Deluxe Plus is designed only to correct slight imbalance when climbing over the side of the bath. It must not be used to pull or hold your entire body weight, for example when sitting down or standing up in the bath – see figure 2!
- Prior to fitting, the side of the bath must be dry and free of grease!
- The width of the edge of the bath must not be less than approx. 8,5 cm as a secure fixture of the aid cannot be guaranteed in this case!
- The Bathtub Safety Rail must not be fitting to baths with plastic outer sides!
- The locking screw should only be unscrewed until the point when the mark »max.« appears on the snap-on mechanism – see Fig. 3!

fig. 1

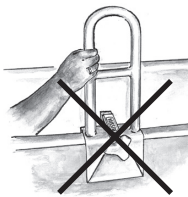
fig. 2

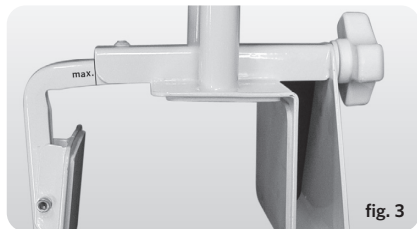
fig. 3

Assembly and fitting

The Bathtub Safety Rail Deluxe Plus is delivered fully assembled. Before the aid is fitted, ensure that the side of the bath is dry and free of grease. The locking screw should only be unscrewed until the point when the mark »max.« appears on the snap-on mechanism – see fig. 3!

Place the Bathtub Safety Rail onto the side of the bath with the fixing screw facing outside, using the screw to fix the entry aid in place so that all three sides of the fixing bracket rest firmly against the side of the bath – see fig. 1.

The three sides of the fixing bracket have a rubber coating to avoid scratching the bath and increasing the stability of the entry aid.

The Bathtub Safety Rail Deluxe Plus fits baths whose edges are between 8,5 cm and 13 cm deep. Use of the Bathtub Safety Rail Deluxe Plus is generally no problem for baths which have been walled in place. Please contact your local bathroom specialist for advice of fitting if the tiles around your bath are not attached to masonry. The Bathtub Safety Rail Deluxe Plus is not suitable for baths whose sides are covered only by a plastic panel, as these cannot guarantee the necessary stability.

Cleaning

Clean the Bathtub Safety Rail using normal household or bathroom cleaner.

Material

Coated steel tube, TPR

Disposal

Disposal of this product in accordance with your regional waste disposal regulations.

Technical details

- Total height: 48 cm approx.
- Total width: 15 cm approx.
- Height above the edge of the bath: 36 cm approx.
- Total weight: 2,6 kg approx.
- Clamp width: 8,5-13 cm approx.
- Maximum load capacity: Under normal use, suitable for persons weighing up to ca. 100 kg
- Maximum stress capacity: 450 N (ca. 45 kg)

This device complies with Regulation (EU) 2017/745 on medical devices and bears a CE marking.

If a serious incident occurs involving this medical device, contact the manufacturer of this device at info@russka.de as well as the competent authority, the Medicines & Healthcare products Regulatory Agency MHRA, www.gov.uk/government/organisations/medicines-and-healthcare-products-regulatory-agency.

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Fabrikant: Ludwig Bertram GmbH, Im Torfstich 7, 30916 Isernhagen, Duitsland, www.russka.de