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Intended use:

The Bathtub Bar is designed to provide secure assistance when entering and exiting the bathtub. It is installed by simply placing it onto the edge of the bathtub and tightening the locking screws. This product must not be used by individuals with limited grip strength, weakness in the upper limbs, dizziness, or insufficient mental or physical capabilities. If necessary, assistance from a caregiver should be obtained.

Assembly:

- Ensure that the area where the Bathtub Bar will be mounted is free of grease and soap residue.
- First, remove the white caps from the locking screws ❶.
- Loosen the locking screws ❶.
- Position the Bathtub Bar onto the bathtub edge (with the shorter feet facing inward).
- Tighten both locking screws ❶ firmly.
- Check the stability of the bar by shaking it or applying pressure.
The device must only be used if it is securely and firmly attached.
- Finally, replace the white caps onto the screw heads.

Not suitable for bathtubs with plastic paneling!**Cleaning and disinfection:**

The Bathtub Bar can be cleaned using common household cleaning agents. Make sure that the cleaner contains no oils or greases. If needed, standard chlorine-free disinfectants may be used.

Important: Always follow the manufacturer's instructions when using disinfectants. The expected service life is approximately 10 years, depending on frequency of use and the condition of the product.

⚠ Safety instructions:

- Caution, product and packaging are no toys, risk of injury and suffocation! Do not leave children unattended with it! Only use the Bathtub Bar for its intended purpose!
- Maximum weight capacity: 100 kg.
- Check the secure fit of the safety rail before each use.
- Do not use oily or greasy bath additives as these increase the risk of slipping!
- Use original replacement parts only.
- Do not use damaged or defective products.

Warranty:

According to the statutory warranty claims. Excluded from the guarantee are defects resulting out of the natural wear and tear, intent, negligence or improper handling or use and defects which affect the value or usability of the tool only insignificantly.

We reserve the right of modifications serving the technical improvement.