Gebrauchsanweisung / User instruction / Bruksanvisning 1-1121

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EN – Generally: For the described article the conformity to the EC Directive 2017/745 for medical devices is confirmed. The manual is part of the article and has to be kept for the user. Also when the user change the instruction remains part of the product. Reprints, translations and copying in any form, even partially, without the express written consent from the manufacturer Nordic activity AB / Sweden not allowed. The copyright lies with the manufacturer. This instruction is not subject for revision. For the current status information, please contact us. Any changes through continued development over the design presented in this manual are without prior notice possible.

Indications: The article was designed for persons with reduced mobility and need for support and assistance.

Safety / use: The article must stand securely on all four feet. After each individual height adjustment the article has to be controlled. For stability must be verified that all the legs are the same length. Slipping risk is when sitting down and/or standing up. Please use if needed to sit down and stand assistance by a helper if you can't keep himself sufficiently. Do not lean extremely forward or backward – TIPPING-RISK! Make sure before use that the article is firmly and securely standing on the Ferrules.

Delivery: The article is delivered in a carton assembled and ready for use.

Cleaning: Cleaning and disinfection you should perform on a regular basis (at user change required). When cleaning, use a mild soap lotion without abrasive additives. The disinfection, you can perform with conventional solutions.

Maintenance: The article is maintenance free. Any broken components – please contact your dealer or your service for replacement.

Warranty: The warranty period is 24 months and covers all defects in the product that is proved to be material or workmanship.

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