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► User Manual

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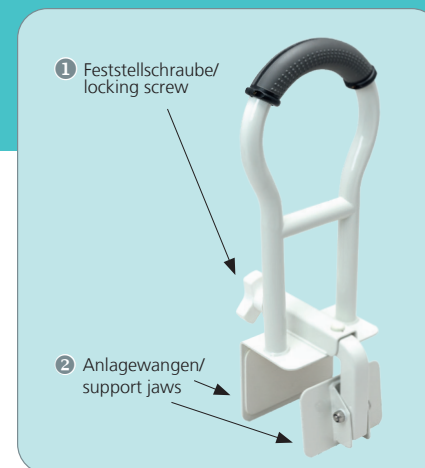
EN

Intended use:

The Bathtub Bar with Soft Grip is a medical device and used to get in and out of the bathtub more safely. It is fastened by simply attaching and tightening the locking screw.

Indication:

Insecurity when overcoming the edge of the bathtub.

Contraindications:

Grip weakness, upper limb weakness, dizziness, no sufficient physical or mental abilities to use the aid.

Versions:

There are different aids to get over the edge of the bathtub more safely. The selection of the appropriate product should be carried out together with the specialized staff. Testing, assembly and instruction in use must only be carried out by authorized, trained specialist personnel. The model is designed for domestic use, must only come into contact with intact, uninjured skin and must not be used across patients. Any use other than that mentioned above is not intended. For resulting damages there is no liability by Sundo GmbH, the risk is borne exclusively by the user. In the event of serious incidents occurring in connection with this product, please inform the manufacturer or the competent authority.

Before the first use

First of all, read the present instruction for use in full and keep it with the packaging until the end of the product's life. Please check before the first and each use by visual inspection whether the product is undamaged. In case of signs of damage or wear, the Bathtub Bar must not be used (any longer).

Assembly:

- The area in which the entry aid is to be installed must be free from grease and soap.
- Loosen locking screw ❶ by turning counterclockwise.
- Then place the Bathtub Bar on the edge of the bath (with the adjustment screw facing outwards).
- Now tighten the entry aid by turning the locking screw clockwise until the support jaws ❷ are firmly in contact with the edge of the tub.
- Now check the secure fit of the Bathtub Bar by shaking, supporting etc., and retighten if necessary. The aid may only be used with sufficient stability.
- **Warning, not suitable for bathtubs with plastic aprons!**

Cleaning and disinfection:

The Bathtub Bar can be cleaned with standard household cleaners, please make sure that it contains no oils and fats. If necessary, standard, chlorine-free disinfectants can be used.

Important: When using disinfectants, the manufacturer's instructions for use must be observed.

The service life is about 10 years, depending on care and frequency of use.

Safety instructions:

- Caution, choking hazard due to packaging (polybag)! The Bathtub Bar is not a toy! Please do not leave children unattended with the packaging or the product.
- The Bathtub Bar must only be used by persons with sufficient mental/physical abilities, if necessary with the support of a carer!
- The maximum load capacity is 100 kg.
- Before each use, the secure seating of the Bathtub Bar must be checked.
- Do not use oily or greasy bath products. Danger of slipping!
- Only use original spare parts.

Warranty:

According to the legal warranty claims. Excluded from the guarantee: Damage due to improper use, normal wear and tear as well as defects that have only insignificant effects on the value or serviceability of the device. Technical changes and attachments as well as improper maintenance will void the warranty and product liability in general. We reserve the right to make changes that are in the interest of technical improvement.