INSTRUCCIONES DE USO **MODE D'EMPLOI** **GERBRAUCHSANWEISUNG** **ISTRUZIONI PER L'USO**
ANVÄNDNINGSDIAGRAM **INSTRUKSJONER FOR BRUK** **KAYTTOOHJEET** **BRUCSANSVINGING** **ODIATIE XPIJIE**

EN DESIGNED TO REDUCE THE INTERNAL LENGTH OF A BATH BY 12 INCHES. ALLOWING SHORTER PEOPLE TO BATHE IN SAFETY. ENSURE BATH IS DRY AND FREE FROM SOAP OR OIL FILM. TEST FOR STABILITY. DO NOT SIT ON THE ASHBY BATH SHORTENER. DO NOT USE ON A BATH WITH A TEXTURED SURFACE. IT IS THE CURVATURE OR SLOPE. ADVISOR TO CARRY OUT AN INITIAL, AND REGULAR, RISK ASSESSMENTS. MAY NOT BE SUITABLE FOR BATHS WITH EXTREME CURVATURE OR SLOPE.

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PH 5 - 9

ASHBY BATH SHORTENER


Welcome to your new Gordon Ellis product.
We hope that it proves a useful aid in your daily life.

SEE INSIDE AND REVERSE FOR FITTING INSTRUCTIONS
AVAILABLE IN THE FOLLOWING LANGUAGES:

EN	FR	DE	IT	ES	NL	PT	DK	SV	NO	FI	EL
MAXIMUM USER WEIGHT	POIDS MAXIMUM DE L'UTILISATEUR	MAXIMALES BENUTZERGEWICHT	PESO MÁXIMO DEL USUARIO	MAXIMUMGEWICHT DE USUÁRIO	MAKS. BRUGERVÆGT	MAXIMUM BRUKERVÆKT	MAKSIMUM BRUKERVÆKT	MAKSIMUM BRUKERVÆKT	MAXIMUM BRUKERVÆKT	MAXIMUM BRUKERVÆKT	MAXIMUM BRUKERVÆKT
190 kg	420 lbs										

EN ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE MANUFACTURER - CONTACT DETAILS BELOW
FR TOUT ACCIDENT GRAVE SURVENANT PENDANT L'UTILISATION DOIT ÊTRE SIGNALÉ AU EC REP - COORDONNÉES CI-DESSOUS
DE JEDER SCHWERE UNFALL WÄHREND DES GEBRAUCHS MUSS EC REP GEMELDET WERDEN - KONTAKTDATEN UNTEN
IT QUALSIASI INCIDENTE GRAVE CHE SI VERIFICA DURANTE L'USO DEVE ESSERE SEGNALATO AL PRODUTTORE - DETTAGLI DI CONTATTO DI SICURTÀ SOTTO
ES CUALQUIER INCIDENTE GRAVE QUE OCORRA DURANTE O USO DEVE SER INFORMADO A EC REP - DETALLES DE CONTACTO A CONTINUACIÓN
NL ELK ERNSTIG ONGEVAL, TIDDENS HET GEBRUIK MOET WORDEN GEMELD AAN EC REP - CONTACTGEGEVENS HIERONDER
PT QUALQUER INCIDENTE SÉRIO QUE OCORRA DURANTE O USO DEVE SER RELATADO AO FABRICANTE - DETALHES DE CONTACTO ABAIXO
DK ENHVER ALVORLIG HENDELSE, DER OPSTÅR UNDER BRUK, SKAL RAPPORTERES TIL TILVERKAREN - KONTAKTOPPLYSNINGER NEDENFOR
SV ALLA SVÄRRE OCH ALLVARIGA HÄNDELSER UNDER ANVÄNDNING MÅSTE RAPPORTERAS TILL TILLVERKAREN - KONTAKTOPPLYSNINGAR NEDAN
NO ENHVER ALVORLIG HENDELSE SOM OPPTÅR UNDER BRUK MÅ RAPPORTERES TIL PRODUSENTEN - KONTAKTOPPLYSNINGER NEDENFOR
FI KAIKISTA KATYON AIKANA TAPAHTUNEISTA VAKAVISTA TAPAHTUMISTA ON KÄBE ZODIAPATA VALMISTAJALLE - YHTEYSTIEDOT ALLA
EL ΑΝΑΦΟΡΕΤΑ ΤΩΝ ΚΑΤΑΒΕΒΛΗΤΩΝ - ΠΑΡΑΚΑΤΩ ΣΤΟΙΧΕΙΑ ΕΠΙΚΟΙΝΩΙΑΣ

Your ASHBY BATH SHORTENER was specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.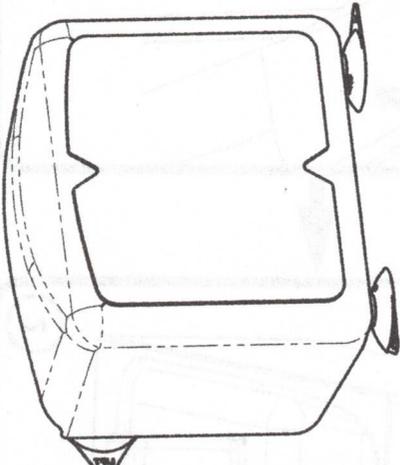

GORDON ELLIS & CO.
TRENT LANE
CASTLE DONINGTON
DERBY, ENGLAND, DE74 2AT





EC REP
AUTHORISED REPRESENTATIVE
ADVENA LTD, TOWER BUSINESS CENTRE, 2ND FLOOR, TOWER STREET, SWATAR, BKR 4013 MALTA

MD **IFU-REF 6290/0920-1** **PART REF 6290**

BHTA **4 MOPE**

VIEW THE ENTIRE RANGE ONLINE AT **gordonellis.healthcare**
GORDONELLISHEALTHCARE.CO.UK

1 FITTING INSTRUCTIONS

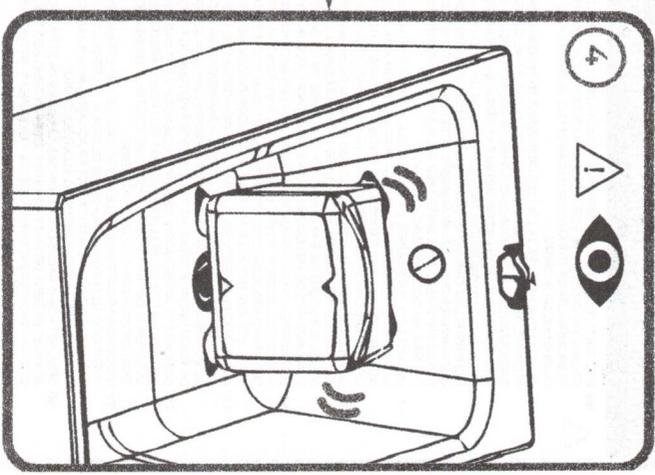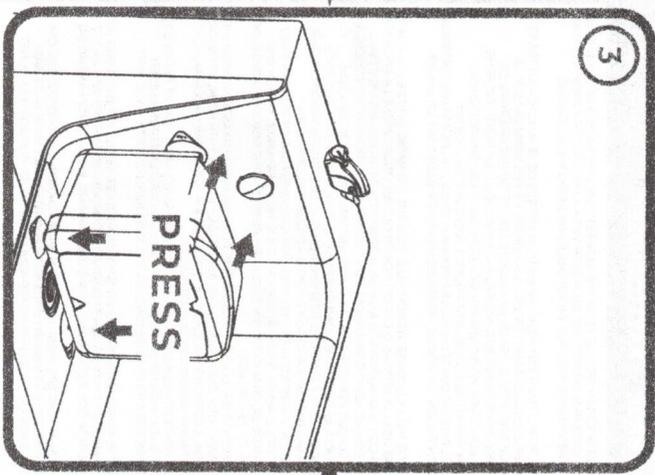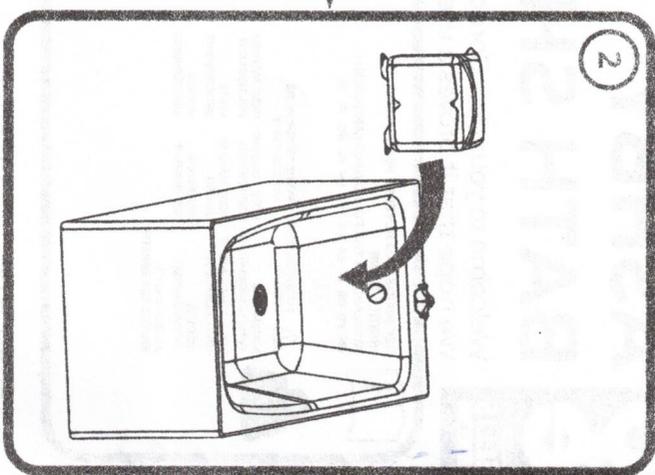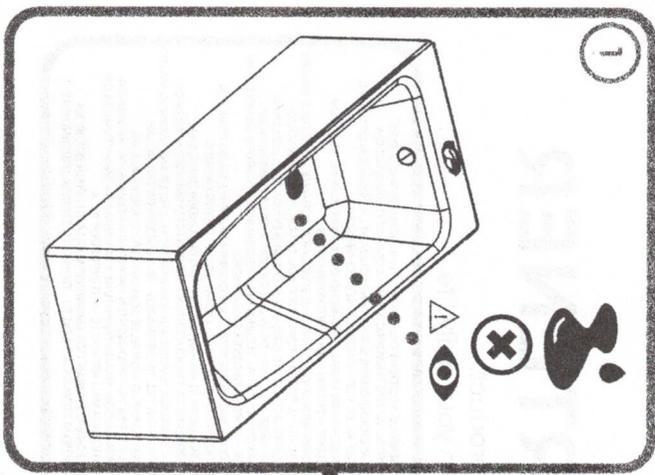1 REMOVAL INSTRUCTIONS